


ADJUSTING TABLET SETTINGS AUTOMATICALLY


To ensure system settings won't be available for unauthorized people we suggest **turning system notification off** and **running system in guest mode**.

All steps described in Appendix B can be done automatically with use of prepared by Amplivox scripts.


1. Open „**Tablet set up**“ folder on your BEEP USB bundle stick and double click on **STEP_1** file.
This step will set up virtual keyboard that will be used by patients to add their data.

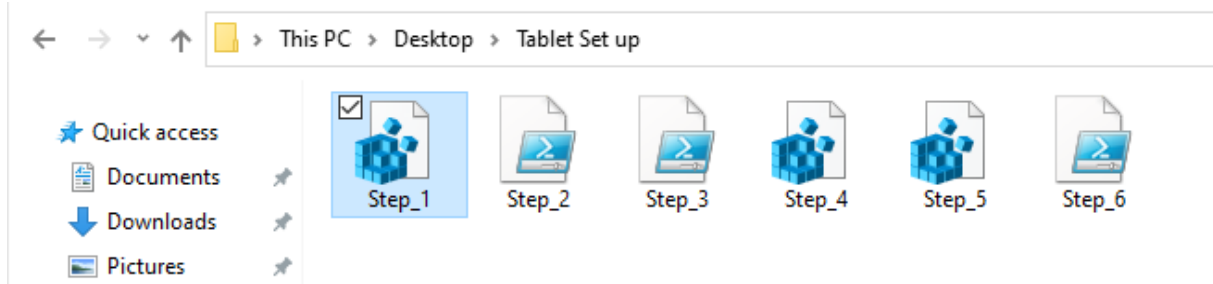


2. Right mouse button click on **STEP_2** file and select “Run with PowerShell”.
This step will restart Explorer.exe process, to prepare system for next steps.

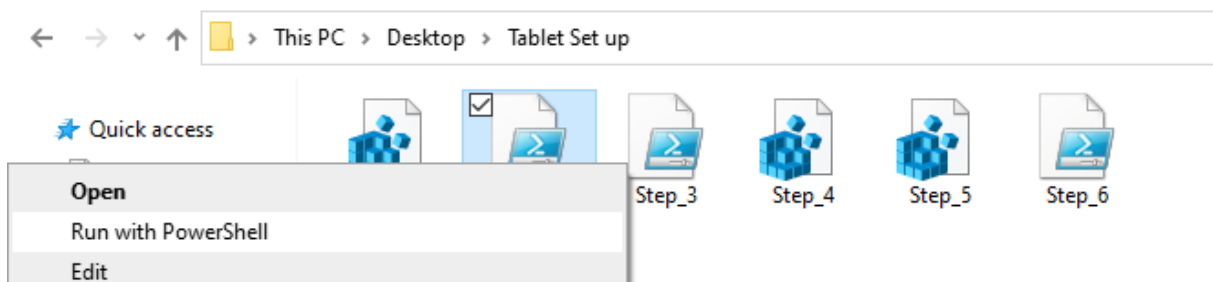

3. Back in the „**Tablet set up**“ folder right mouse button click on **STEP_3** and select “Run with PowerShell”.
This step will create **Guest account** with password (when password is not required password field should be left empty) and **restart the tablet**.

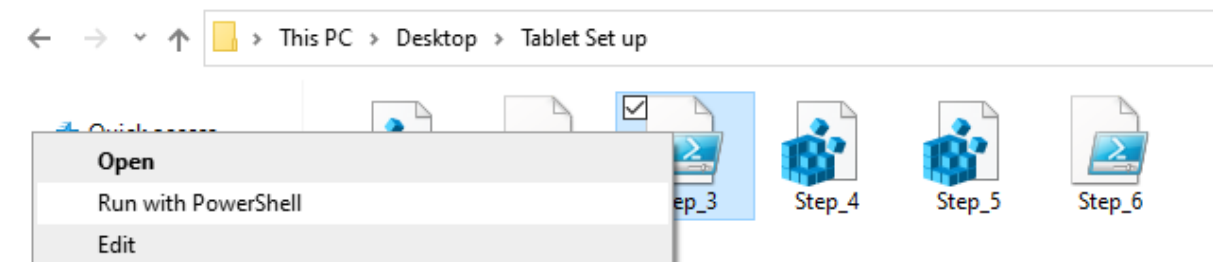


4. After system restarts, enter **Guest** account and back in the „**Tablet set up**“ folder, double click on **STEP_4** file.
This step will set up virtual keyboard but on the Guest account.

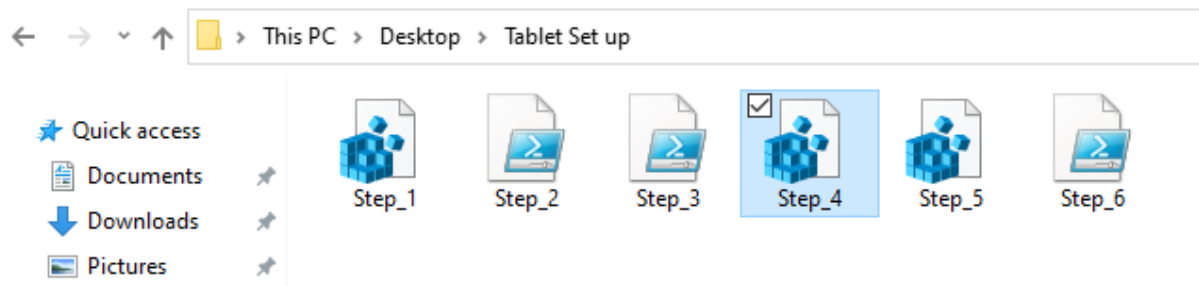

5. Double click on **STEP_5** file.

This step will turn OFF changing screen position, so users won't be able to close BEEP app, in not intended way.

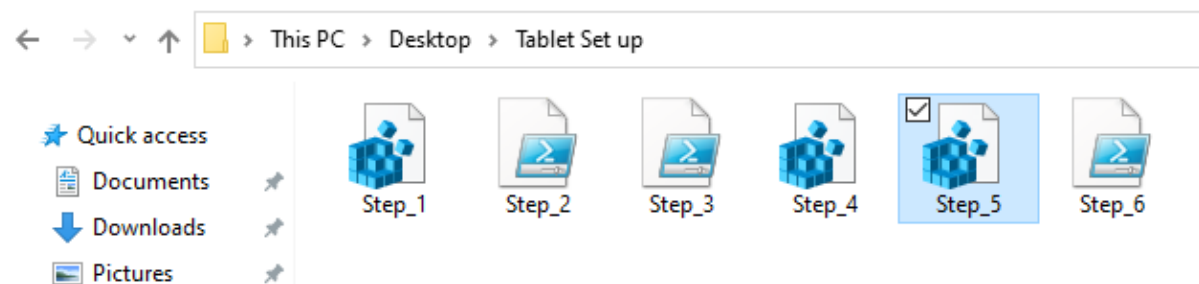

6. Log off from Guest account. Log back in your main account. Back in the „Tablet set up“ folder right mouse button click on **STEP_6** and select “Run with PowerShell”.

This step will change **Guest** account rights from **Admin** to **Visitor**.

